



Portable Freshness

Oral Appliance Device Cleaner

Colour: Light Blue

Scent: Mint



www.micrylium.com/warranty

Available Sizes:

60 mL Travel Sprayer (case of 10) (05-MOFF-060)

Purpose:

MouthOFF™ cleans and refreshes oral appliances without staining, while minimizing the impact on the Environment, People and Equipment.

Suitable for:

Cleaning and protecting oral appliances / devices including dentures, sleep apnea machines, mouthguards, soft plastics, metals, silicone and acrylics. Compatible with Nebulizers, Aerochamber® and other devices with silicone masks / mouthpiece.

Applied Cleaning Concepts:

MouthOFF™ contains biodegradable surfactants for cleaning and protecting oral appliances.

Quality Commitment:

Our vision of cleaning holistically links the patient/client safety with the safety of professional staff. Our chemistry is based on materials of pharma-grade (USP EP BP NF) origin. Quality Ingredients include biodegradable surfactants, chlorhexidine, xylitol with USP grain alcohol and high purity water. OECD 301D Tests conclude biodegradability (28 days).

Instructions for Use on Oral Appliances:

Rinse the device thoroughly. Spray or submerge the device for a minimum of 5 min. Rinse the device again before use.

Instructions for Use on Sleep Apnea Machines:

- 1) Unplug your CPAP machine from the power source.
- 2) Disassemble your mask into 3 parts (headgear, cushion and frame).
- 3) Cushion should be cleaned daily using MouthOFF™. Spray MouthOFF™ generously on the cushion and rinse thoroughly. Dry cushion with a clean Micrylium LeCLOTH™. Headgear and frame should be cleaned weekly with Micrylium BioSON™. In a sink or tub, clean your mask cushion and headgear to remove any oils. Gently rub with a mild liquid detergent and warm, drinking-quality water. Avoid using stronger cleaning products as they may damage the mask or leave harmful residue.
- 4) Use BioSON™ at a dilution rate 1:200 which is 5mL of solution for 1L of potable water in a sink or tub to clean and remove oils from the mask cushion and headgear. Gently scrub the appliance in the solution with a saturated LeCLOTH™. Simultaneously immerse the air tubing in the solution.
- 5) Rinse all components thoroughly with warm potable water being sure to remove all residue.
- 6) Place on a flat surface, on top of a towel, to dry. Use a dry LeCLOTH™ to remove any moisture from irregular areas where it may pool. Avoid placing in direct sunlight.

Precautions:

USE FULL STRENGTH – Do not dilute. RTU (Ready To Use Formulation)

Contains 9.5% Ethanol and Xylitol. (Keep away from Pets)

Helpful Tips:

To ensure good results, rinse the device thoroughly before application.

See our website for specific details: www.micrylium.com/products



Revision Date: Dec 15, 2023 v2.0

